



- APPROPRIATE SETTINGS FOR THE INSERTS
- SETTAGGI APPROPRIATI PER GLI INSERTI
- KORREKTE EINSTELLUNGEN FÜR DIE INSTRUMENTE
- RÉGLAGES APPROPRIÉS POUR LES INSERTS
- AJUSTES APROPIADOS PARA LOS INSERTOS
- KORREKTA INSTÄLLNINGAR FÖR SPETSARNA

→ MULTIPIEZO touch → COMBI touch → MICROPIEZO



function	endo perio/scaler restorative	endo perio/scaler restorative	endo perio-scaling prosthetics
----------	-------------------------------	-------------------------------	--------------------------------

SCALING

S1-S	1-2 **	1-2 **	1-2 **
S1, S2, S3, S4, S5, S6	1-6 **	1-6 **	1-3 **

PERIO

P1, P2, P3, P4, P10, P11, P12, P13, P14	1-3 **	1-3 **	1-3 **
P15, P16R, P16L	1-3 **	1-3 **	1-3 **

IMPLANT CLEANING

ICS	1-2 **	1-2 **	1-2 **
-----	--------	--------	--------

ENDO

E1, E2	1-6	1-5	1-2 **
--------	-----	-----	--------

ENDO REVISION

ER1	1-6	1-5	1-3
ER2, ER3, ER4, ER5	1-3	1-3	1-3

CROWN PREP

DB1	4-6	4-5	0 **
-----	-----	-----	------

EXTRACTION

ME1, ME2, ME3 ***	6	5	*
-------------------	---	---	---

* do not use with this device
 ** can be used with or without „soft mode“ function
 *** not available in the United States and in Italy